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Art. 146P, 1461P, 1478P, 1481, 1479, 1482

## General product instructions

The Toolflex crutch and walking stick holder is used for simple and safe fixation of walking aids and thus makes everyday life easier for people with limited mobility

The Toolflex holder is characterized by the following features:


- Elastic EPDM rubber in the holder part holds the walking aids securely.
- Easy application without great effort
- Highly loadable and insensitive to vibrations.
- Support rail or single mounting for attachment to the wall.

## Mounting art. no. 146 - Toolflex-Holder for the wall

Fixing a support rail with several support elements to the wall:


1. Remove the black caps at the ends of the support rail and, if necessary, pull the holders sideways off the rail (see illustrations).
2. Find a suitable place on the wall. Place the support rail in the desired position and mark the drill holes.
3. Drill the holes at the marked positions. Make sure beforehand that there are no supply pipes or electrical cables at the drilling points!
4. Fix the rail to the wall. To do this, use the appropriate dowels and screws that are suitable for the construction of the wall.
5. Now push the holders back onto the support rail.
6. Finally, put the cover caps back on the ends of the rail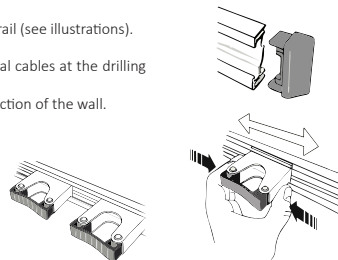

## Mounting art. no. 1461 / 1478 - Toolflex-Holder for single mounting

Fixing a single holder element to the wall or to another flat surface:


1. First remove a locking pin from the holder and pull the holder element forward (see illustration).
2. Find a suitable place on the wall. Place the crutch and walking stick holder in the desired position and mark the drill holes.
3. Drill the holes at the marked positions. Make sure beforehand that there are no supply pipes or electrical cables at the drilling points!
4. Attach the holder to the wall. To do this, use the appropriate dowels and screws that are suitable for the construction of the wall.
5. Turn the holder element back to its original position and insert the locking pin back into the opening.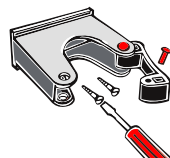

## Use

Walking aids and sticks can be pressed into the device holders with a simple movement, secured and removed from the holder again without much effort.

Always check that the holders are securely fastened before each use. Likewise, always check the firm hold of the walking aid in the holder each time it is used.

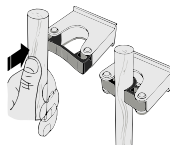## Cleaning

Cleaning and disinfection can be performed with commercially available products.

## Warranty and liability

You have purchased a high-quality Ossenberg product. If, despite the highest quality standards, a justified material defect should occur, Ossenberg has the right, within the framework of a 12-month warranty, to repair the defective item or to replace it with a defect-free subsequent delivery, at its own discretion. We cannot accept any liability for damage caused by modifications to our products. Not covered by the warranty are defects that are due to wear or improper handling. Wear parts are excluded from the warranty.

## Disposal

Please ask your specialist dealer about the regional disposal regulations.

## Technical data

### Toolflex-Holder for the wall

Article	146P	1461P	1478P
Title	Holder for the wall	Holder for single mounting	Holder for single mounting
Tube- $\phi$	from 20 mm to approx. 30 mm	from 20 mm to approx. 30 mm	from 30 mm to approx. 40 mm
Other	incl. support rail		

### Support rails for Toolflex-Holders (without holders)

Article	1481	1479	1482
Length	20 cm	50 cm	90 cm
Other	Mounting material is included.		

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Ossenberg GmbH